



AN ITALIAN COMPANY THAT RESEARCHES, DEVELOPS AND PRODUCES READY TO USE KITS
USING CHROMATOGRAPHIC TECHNIQUES FOR CLINICAL LABORATORIES

CONTROL LIOPHYL. IN PLASMA FOR VITAMIN C - LEVELS 1 and 2

Code Z20019

OBJECTIVE

These controls in plasma are used for the internal quality control and serve to monitor the accuracy and precision of analytical procedures devoted to the quantitative determination of analytes contained therein. These controls are lyophilized human matrix and are available in two different concentration ranges. Should be handled as if they were a real patient sample.

RECOVERY

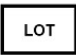


Remove the metal seal and rubber stopper from the vial. Add exactly 1 ml of HPLC-grade water through the tube. Replace the rubber stopper, allow to stand for at least 5 to 10 minutes. Take 900 μ l and add 100 μ l of **Reagent B - Reducing Solution** (after reconstitution as per method). Before use, mix by inverting the vial to dissolve the material until a homogeneous solution.

STORAGE AND STABILITY

The controls are stable for 36 months from the date of preparation if stored at -20° C. After reconstitution are stable for 1 day at $2 - 8^{\circ}$ C and 3 months at -20° C. Do not use after the expiry date.

PRECAUTIONS

These controls in human matrix should be treated with care and treated as potentially infectious.

		
010	May 2023	May 2026

PACKAGING AVAILABLE:




- Z20019 CONTROL IN PLASMA FOR VITAMIN C - LEVELS 1 and 2

2 x 5 x 1 ml

Release N° 002

September 2019

CONCENTRATIONS
CODE Z20019

		
010	May 2023	May 2026

<i>ANALYTE</i>	<i>MEASUREMENT UNIT</i>	<i>MEDIUM VALUE L1</i>	<i>RANGE L1</i>	<i>MEDIUM VALUE L2</i>	<i>RANGE L2</i>
VITAMIN C	mg/l	4,5	3,2 – 5,9	33,3	23,3 – 43,3

This product fulfills all the requirements of Directive 98/79/EC of 27/10/1998 on in vitro diagnostic medical devices (IVD). The declaration of conformity is available upon request.



FOR IN VITRO DIAGNOSTIC USE ONLY

produced by



AN ITALIAN COMPANY THAT RESEARCHES, DEVELOPS AND PRODUCES READY TO USE KITS
USING CHROMATOGRAPHIC TECHNIQUES FOR CLINICAL LABORATORIES

EUREKA srl – LAB DIVISION ,
Via Enrico Fermi 25, 60033 Chiaravalle (AN) ITALY
Tel. + 39 071 74 50 790
Fax. + 39 071 74 96 579
info@eurekaone.com
www.eurekakit.com



COMPANY WITH
MANAGEMENT SYSTEM
CERTIFIED BY DNV
= ISO 9001 =
= ISO 13485 =

Release N° 002

September 2019