



AN ITALIAN COMPANY THAT RESEARCHES, DEVELOPS AND PRODUCES READY TO USE KITS  
USING CHROMATOGRAPHIC TECHNIQUES FOR CLINICAL LABORATORIES

## LYOPHILIZED CONTROL FOR CITRATES/OXALATES IN URINE - LEVELS 1 and 2

**Code Z32219**

### PURPOSE

These controls in urine are used for the internal quality control and serve to keep under control the accuracy and precision of the analytical procedures dedicated to the quantitative determination of the analytes contained therein. These controls are in lyophilized human matrix and are available in two different concentration ranges.

### RECONSTITUTION



Remove the metal seal and rubber stopper from the vial. Add exactly 5 ml of HPLC grade H<sub>2</sub>O in the vial. Replace the rubber stopper, shake and let stand for 5 to 10 minutes. Before use, mix by inverting the vial to dissolve the material until a homogeneous clear solution.

### STORAGE AND STABILITY

The controls are stable for 36 months from the date of preparation if stored at 2-8 ° C. After reconstitution are stable 7 days at 2-8 ° C and 1 month at -20 ° C. Do not use after the expiry date.

### PRECAUTIONS

These controls in human matrix should be treated with care and treated as potentially infectious

LOT		

### PACKAGING AVAILABLE:

- Z32219 CONTROL FOR CITRATES/OXALATES IN URINE - LEVELS 1 and 2

2 x 5 x 5 ml

N° 003

07/2024

CONCENTRATIONS  
CODE Z32219

LOT		

<i>ANALYTE</i>	<i>UNIT OF MEASUREMENT</i>	<i>MEDIUM VALUE L1</i>	<i>RANGE L1</i>	<i>MEDIUM VALUE L2</i>	<i>RANGE L2</i>
CITRATES	mg/l				
OXALATES	mg/l				

This product fulfils all the requirements of Directive 98/79/EC of 27/10/1998 on *in vitro* diagnostic medical devices (IVD). The declaration of conformity is available upon request.



**FOR IN VITRO DIAGNOSTIC USE ONLY**



EUREKA S.R.L. LAB DIVISION  
Via M. D'Antona 28  
60033 Chiaravalle (AN) - Italy  
Tel +39 071 7450790  
eureka-support@sentinel.it  
www.eurekakit.com